VETIGEL®

PRECISION ACCESSORIES

Rx ONLY, ANIMAL-USE ONLY. NOT FOR HUMAN USE.

INSTRUCTIONS FOR USE

This box includes sterile VETIGEL® Precision Accessories for use with VETIGEL® Haemostatic Gel syringes. VETIGEL® Haemostatic Gel syringes are sold separately.

Caution: For animal use only. Not for human use. This device is restricted for use by or on the order of a veterinarian.

DESCRIPTION

Each VETIGEL® Precision Accessory pouch contains two dispensing tips and one syringe adaptor. The dispensing tip and syringe adaptor are intended for use with VETIGEL® Haemostatic Gel syringes. Each pouch is terminally sterilised with gamma irradiation.

Dispensing Tip: The VETIGEL® Accessory Dispensing Tip is a 45-degree angled blunt tip dispensing needle with a luer connector intended to allow targeted application of VETIGEL® into the wound site. The dispensing tip is composed of 304 stainless steel and polypropylene.

Syringe Adaptor: The VETIGEL® Accessory Syringe Adaptor is a VETIGEL® syringe thread to luer connector adaptor that allows the use of a variety of accessories with the VETIGEL® syringe. The syringe adaptor is composed of biocompatible polypropylene resin.

INTENDED USE/INDICATIONS

VETIGEL® Precision Accessories, in conjunction with VETIGEL® Haemostatic Gel, is intended for animal use only.

CONTRAINDICATIONS

- Reference VETIGEL® Haemostatic Gel Instructions for Use for additional contraindications.

WARNINGS

- Avoid abrasion of wound sites with open end of the dispensing tip.
- Reference VETIGEL® Haemostatic Gel Instructions for Use for additional product warnings.

PRECAUTIONS


- VETIGEL® Precision Accessories can only be used by or under the direction of a veterinarian.

- Do not use VETIGEL® Precision Accessories if the packaging seal is torn and/or damaged.
- VETIGEL® Precision Accessories must not be used beyond its expiry date.
- VETIGEL® Precision Accessories are manufactured and supplied as one-time use medical devices. They cannot be sterilised before use or after contents have been opened or exposed.
- Reference VETIGEL® Haemostatic Gel Instructions for Use for additional precautions.

DIRECTIONS FOR USE


- Retrieve one VETIGEL® Haemostatic Gel syringe pouch (sold separately). Though VETIGEL® Haemostatic Gel does not require preparation, it is recommended that prior to beginning surgery, one VETIGEL® Haemostatic Gel syringe pouch is pre-emptively taken out of the box and kept readily accessible in case of need
- Remove the syringe, dispensing tip and syringe adaptor from their pouches.
- When ready to use, unscrew syringe cap and screw the syringe adaptor to the syringe using a clockwise motion until secure. Screw the dispensing tip to the syringe adaptor. Do not overtighten.

Device Assembly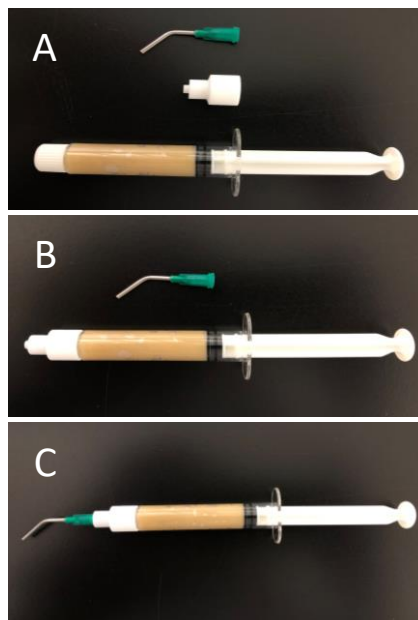


- Prime syringe by slowly depressing plunger until gel reaches the tip of the applicator. Wipe any excess gel that comes out from the nozzle with sterile gauze.

Prime the Assembled Device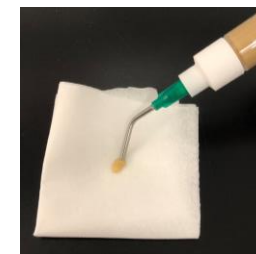


- Identify the source of bleeding.
- Clear as much blood as possible from wound site prior to application. This can be achieved with saline irrigation and/or cotton-tipped applicator. This will allow the hemostatic gel to come in direct contact with the source of bleeding when applied.
- Fill the defect site with the gel, starting at the deepest part and withdrawing the syringe slowly as the wound cavity fills. At the end of the application, when the gel is flush with the surrounding tissue, shear the gel by bringing the nozzle of the syringe away from the defect and downward toward the adjacent tissue in a wiping motion to ensure the gel does not adhere to the applicator tip.
- Closure of the tissue over the gel is not recommended in dental procedures.

CORRECT Haemostatic Gel Application For Dental Applications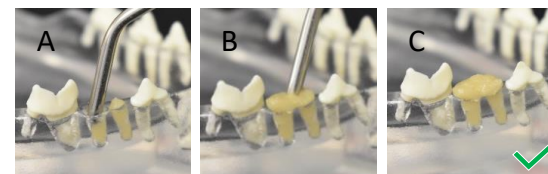

CORRECT Haemostatic Gel Application for Small Wound Sites

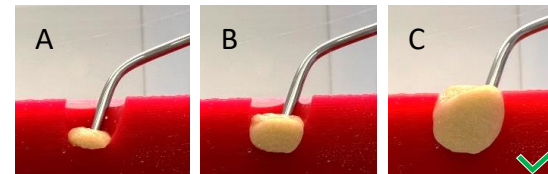Caution: Strong bleeding after haemostatic gel application may be due to misapplication. If so, follow the instructions below.

In the event of misapplication:

Use a blunt probe and/or cotton-tipped applicator to remove gel from the wound site. If there are still remnants of gel within the defect, use saline lavage to remove the prior application. Repeat Directions 5, 6, & 7 until haemostasis is achieved.

STORAGE AND HANDLING

- VETIGEL® Precision Accessories should be stored in a cool dry place.
- VETIGEL® Precision Accessories are for single use only. It is recommended that VETIGEL® Precision Accessories be used as soon as the package is opened.
- Reference VETIGEL® Haemostatic Gel Instructions for Use for storage and handling procedures of VETIGEL® Haemostatic Gel syringes.

MANUFACTURER

VETIGEL® Precision Accessories are manufactured by Cresilon, Inc., a New York City-based medical device company:

Cresilon, Inc.
86 34th Street
Suite D603/D604
Brooklyn, NY 11232
Main: +1 800-VETIGEL (+1 800-838-4435)
Email: customerservice@cresilon.com
www.cresilon.com

VETIGEL® and Cresilon® are registered trademarks of Cresilon, Inc.

DEFINITIONS OF SYMBOLS

Symbol	Definition
	Use-by-date (YYYY-MM-DD)
	Consult Instructions For Use
	Caution (consult Instructions For Use)
	Manufacturer
	Date of manufacture
	Batch code
	Sterile
	Do not re-use
	Do not re-sterilize
	Do not use if package is damaged
	US Federal Law restricts this device to sale by or on the order of a veterinarian.

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VETIGEL® och Cresilon® är registrerade varumärken som tillhör Cresilon, Inc.